

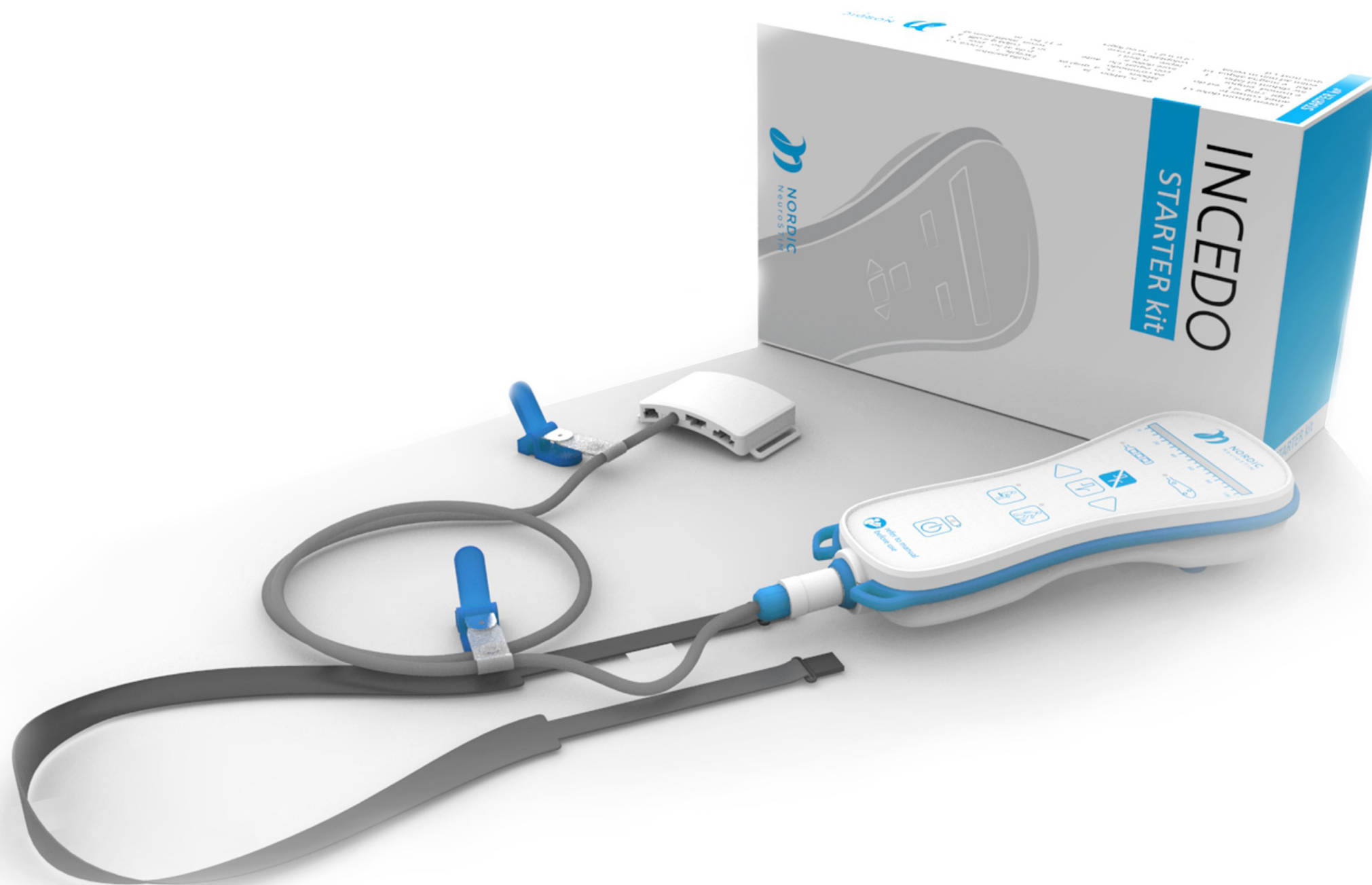
Nordic Neuro STIM Aps

Incedo - koncept design

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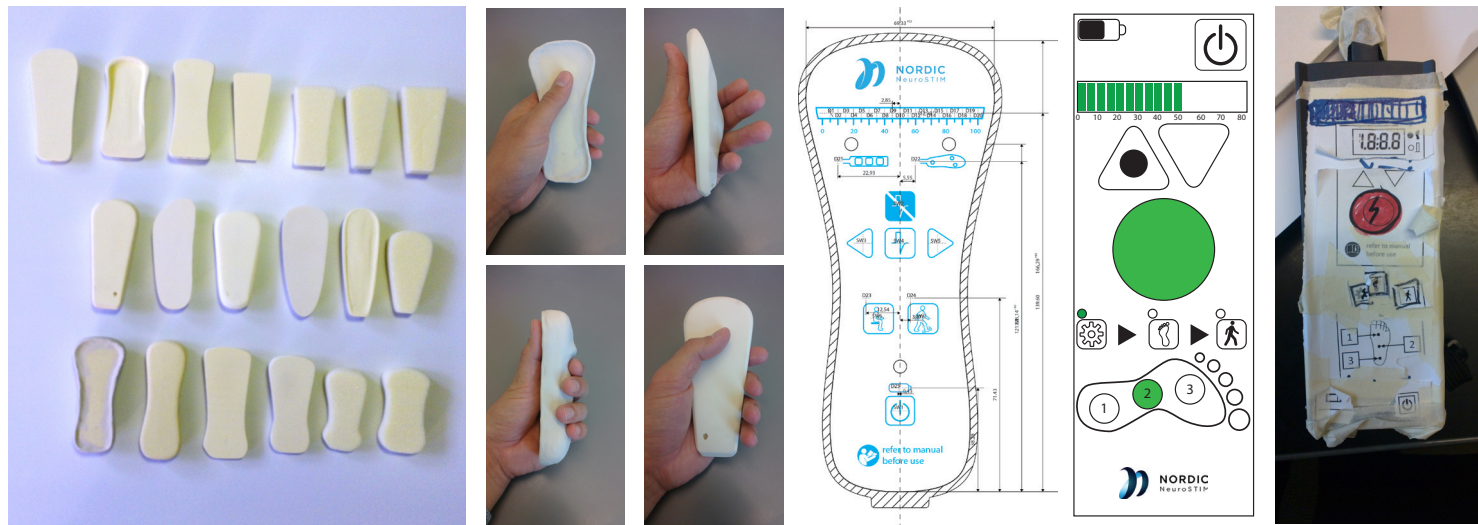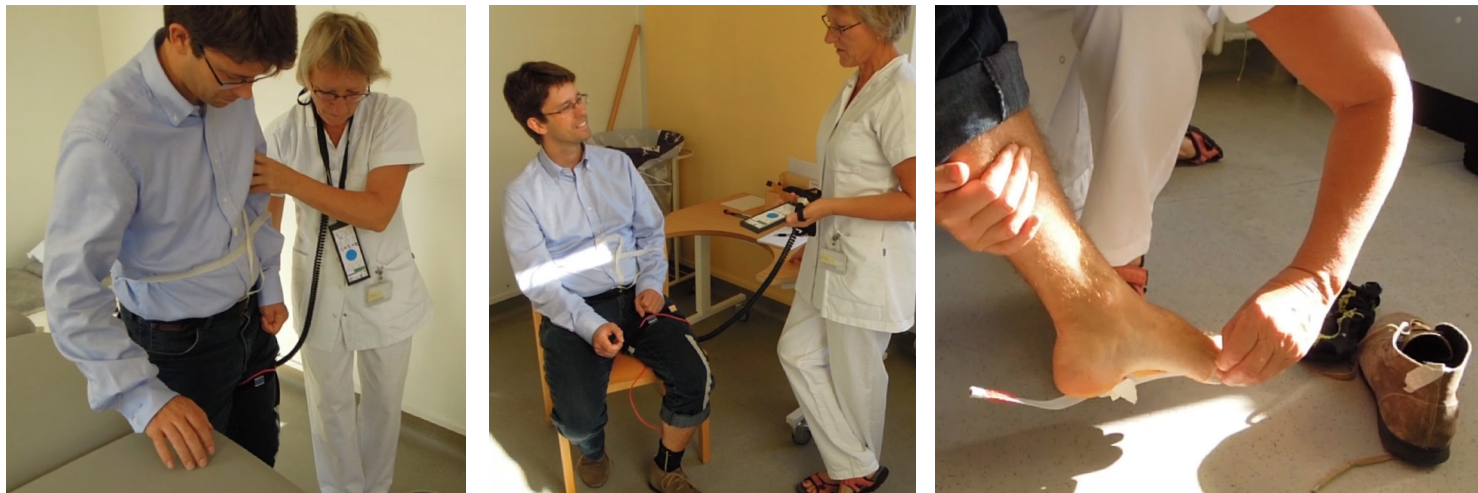OVERVIEW

Reflex-based electrical therapy improves the outcome of gait training leading to patients having more independence and a higher quality of life. The method **augments existing gait training methods** without requiring significant changes to clinical practice.

The method assists patients in walking by **evoking the withdrawal reflex** with electrical stimuli applied to the foot sole. The reflex is

evoked when the patient lifts the foot from the floor, and is controlled by a sensor that is placed in the patient's footwear.

The method is **intended for hemiplegic stroke patients** in their first therapy sessions in order to get them walking sooner and better.

FEATURES

- **Better** and faster gait training.
- **Augments** existing gait training methods.
- **Simple** to learn and use.
- **Portable** and easy to handle.

ADDITIONAL INFORMATION

The system is currently **under development** and will be available for clinical use in **the third quarter of 2015**.

INCEDO SYSTEM

STARTER kit

Reusable parts of the **INCEDO** system:

- Control unit.
- Patient cable.
- Neck strap.

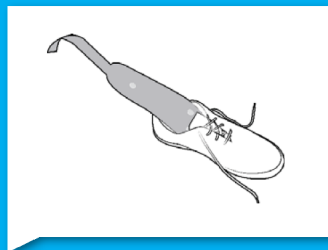
SESSION kit

Disposables that are needed for a gait training session:


- Foot sensor.
- Electrode.

REFLEX-BASED ELECTRICAL THERAPY

FOOT SENSOR

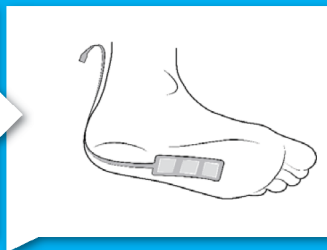The foot sensor is placed in the patient's footwear. It contains multiple sensors distributed over the foot sole. The optimal sensor for gait detection is automatically selected.


INTEGRATED ELECTRODE

A single integrated electrode is placed on the patient's foot sole. The electrode consists of both the active and return electrode for evoking the withdrawal reflex.

SETUP

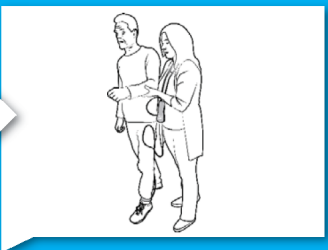The foot sensor and the electrode are connected to the patient cable. The therapist then selects a stimulus intensity that evokes the withdrawal reflex.

TRAINING

Gait training is performed in combination with:

- Over-the-ground walking.
- Weight-reduced treadmill training.
- Other similar existing methods.

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